GREEN SHIELD
THE POWER OF NATURE

SoftMAG ANKLE SHIELD

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customerservice@green-shield.eu

Direttiva CEE 93/42
Dispositivo Médico
Medical Device
Directive EEC 93/42

Swiss antimicrobial expertise
since 1935

Congratulations!

With your purchase you have decided on a high-quality product.
Get to know the product before you start to use it.
Carefully read the following instructions for use.
Use the product only as described and only for the given areas of application.
Keep these instructions safe.
When passing the product on to a third party, always make sure that the documentation is included.
This product conforms to medical device guideline 93/42/EWG.

Contents

1 x Ankle Band
1 x instructions for use

Technical data

Dimensions: 25cm x 28,5cm x 1cm
Weight: 70gr.

Advantages

- Soothes:
- It relieves pain and muscle stiffness in the ankle area.
 - Increases muscle and joint strength.
 - It protects from cooling and can reduce the risk of injury.
 - It stabilizes your posture and can reduce pain.

Application of "Ankle Shield"

Wear Ankle Shield on your ankle while sitting, resting your foot on the floor or a stool. Position the band opening on the heel. Wrap the lower part of the band around the foot. Then wrap the upper part of the band around the leg. Adjust the Velcro fasteners so that the band is taut but comfortable to wear.

Cleaning and maintenance

Before first use hand wash with water (maximum 30 ° C) and mild soap.

Disposal instructions

Dispose of the item compatibly with the environment at an authorized disposal company or at your municipal waste disposal center. Observe the regulations currently in force.

Restrictions on use

Use the product only if it is in perfect condition and in the specified fields of application, as described in this manual.

The product must be used exclusively on intact skin and free of ecchymoses or wounds.

To avoid allergic reactions, avoid the simultaneous use of gels, ointments, oils or similar substances.

Do not use (or only after talking to your doctor) in case of:

- uncertainty regarding use
- Open wounds
- skin problems
- Keep away from metals.
- Keep away from electromagnetic sources (radio, TV, mobile phones, PCs, etc.).
- To patients with high fever
- To pregnant or lactating women
- near metals or electrical systems / devices (even inside the body as metal prostheses)
- to pacemaker wearers.

Warranty

3 year warranty

The product has been manufactured with great care and under constant control. You will receive 3 years warranty on this product from the date of purchase. Please keep the invoice or receipt. The warranty is valid only for defects in material and workmanship and lapses in the event of improper or inadequate use. Your statutory rights, in particular the warranty rights, are not limited by this. In the event of any complaints, send an e-mail to the assistance address below. Our service staff will arrange further interventions with you as soon as possible. In any case we will provide you with personalized advice. The warranty period is not extended by any repairs under the warranty, legal guarantee or courtesy. After the warranty expires any repairs are subject to payment.

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Swiss antimicrobial expertise
since 1935

Material composition

Lycra, MTP5/40, Jersey
28% Polyammide
7% Elastomer
35% Polyester
30% Cotton

Manufactured by
CENTRO STILE
di Lombardi Vincenza
Via Sant'Antonio, 26
76121 - BARLETTA -BT-
ITALY
P. IVA 07434560723
info@green-shield.eu

Production/Production: 2019-05

Version d'impression/Print version: 2019-05

SMAS-0119